MANUALE D'USO
Dr's Viewer 2 ®

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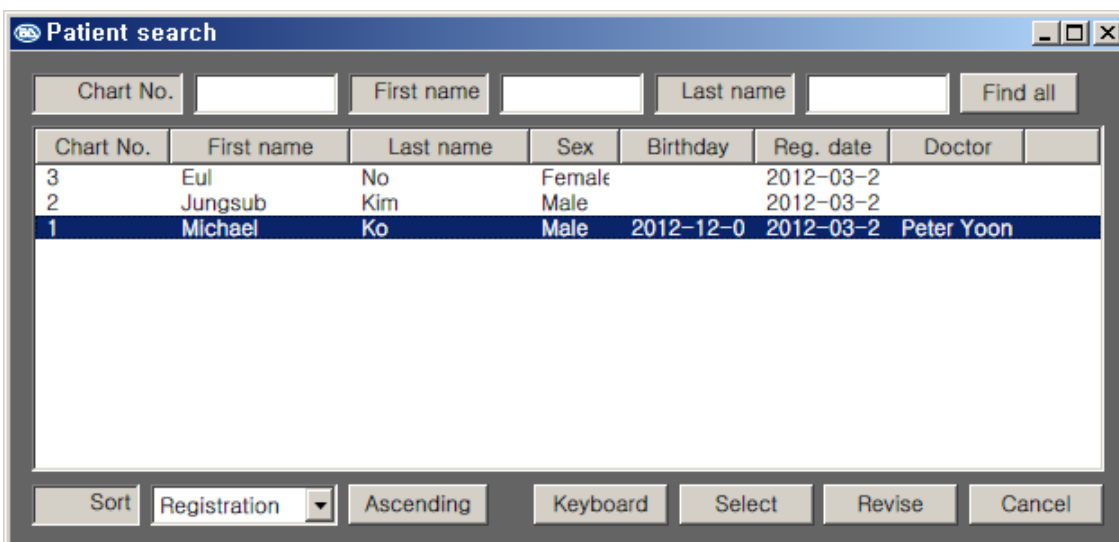
A window titled "Patient search" with search fields for "Chart No.", "First name", and "Last name", and a "Find all" button. Below is a table of patient records.

Chart No.	First name	Last name	Sex	Birthday	Reg. date	Doctor
3	Eul	No	Female		2012-03-2	
2	Jungsub	Kim	Male		2012-03-2	
1	Michael	Ko	Male	2012-12-0	2012-03-2	Peter Yoon

At the bottom of the window are buttons for "Sort", "Registration" (with a dropdown arrow), "Ascending", "Keyboard", "Select", "Revise", and "Cancel".

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A control panel with a "Sort" button, a "Registration" dropdown menu, and an "Ascending" button.

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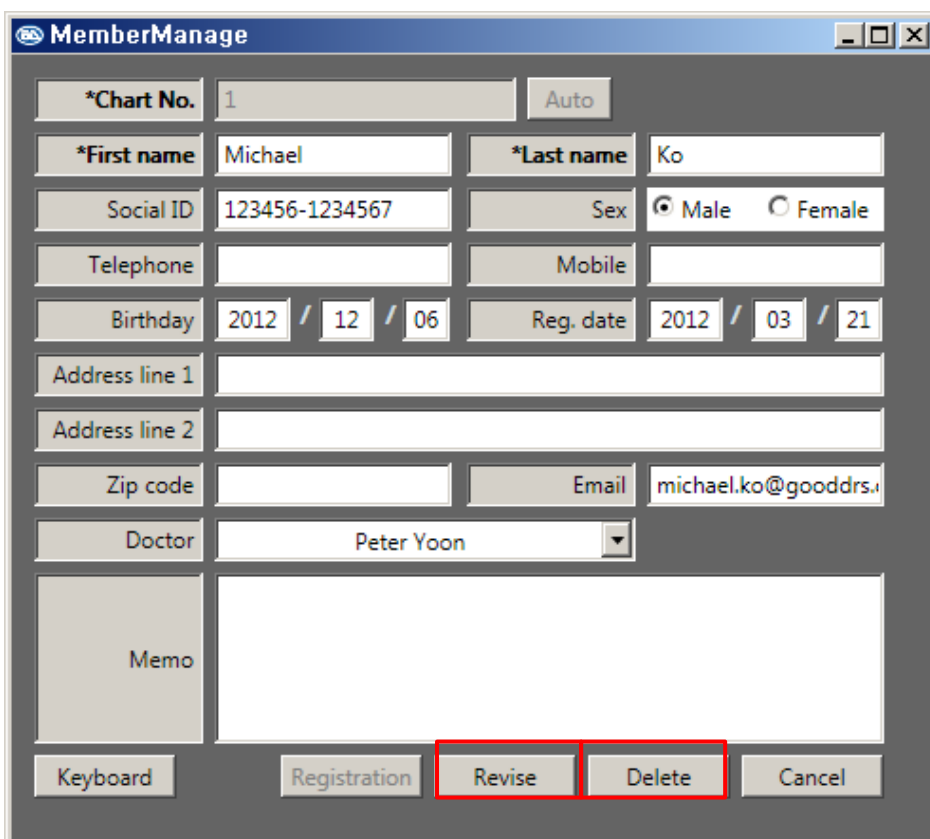
The screenshot shows the 'MemberManage' window with the following fields and controls:

*Chart No.	1	Auto		
*First name	Michael	*Last name	Ko	
Social ID	123456-1234567	Sex	<input checked="" type="radio"/> Male <input type="radio"/> Female	
Telephone		Mobile		
Birthday	2012 / 12 / 06	Reg. date	2012 / 03 / 21	
Address line 1				
Address line 2				
Zip code		Email	michael.ko@gooddrs.	
Doctor	Peter Yoon			
Memo				
Keyboard	Registration	Revise	Delete	Cancel

The 'Revise' and 'Delete' buttons are highlighted with a red border.

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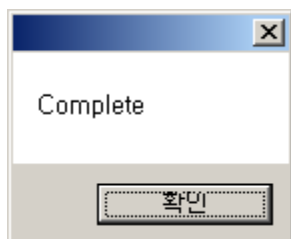
Dr's Viewer 2 / Image software

USER MANUAL
Dr's Viewer 2 ®

*IMAGES AND VIDEOS ACQUISITION
AND VIEWING SOFTWARE*

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

Contents

<u>Chapter 1. Before you begin your program</u>	<u>(04)</u>
<u>Chapter 2. Installation</u>	<u>(05)</u>
<u>Chapter 3. Overview</u>	<u>(08)</u>
<u>Chapter 4. Setting</u>	<u>(10)</u>
<u>Chapter 5. Icon view</u>	<u>(11)</u>
<u>Chapter 6. Patient information view</u>	<u>(20)</u>
<u>Chapter 7. Tap view</u>	<u>(22)</u>
<u>Chapter 8. Image view</u>	<u>(23)</u>
<u>Chapter 9. Main view</u>	<u>(23)</u>
<u>Chapter 10. Data backup</u>	<u>(26)</u>
<u>Chapter 11. Import backup</u>	<u>(27)</u>

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

Chapter 1. Before you begin your program

1. System requirement


OS	Windows XP (32 bit) (SP2~) Windows Vista (32 / 64 bit) Windows 7 (32 / 64 bit) - Operation on Mac OS cannot be supported. - Operation on tablet PC cannot be supported.
CPU	2.0 Ghz CPU
RAM	1 GB
Display	1024 x 768 or greater
HDD	10 GB free space
USB	USB 2.0 or more

Dr's Viewer 2 Operation Instruction


Dr's Viewer 2 / Image software

Chapter 2. Installation

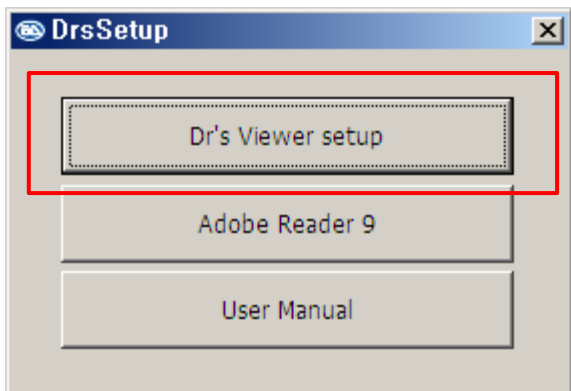
1. Put the USB drive and click “Dr’s Viewer setup”

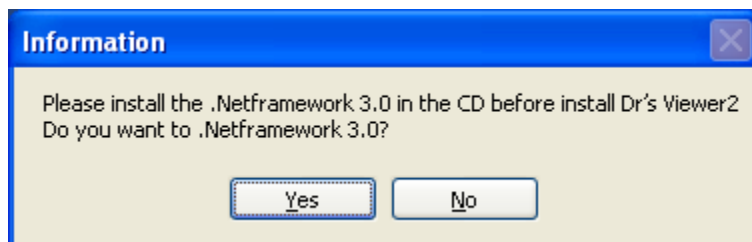2. If .NetFrameWork 3.0 or more is not installed, below message is shown and the installation of .NetFrameWork 3.0 will start. If the program is already installed, the message will not be shown. When .NetFramework 3.0 installation is complete, repeat step 1.

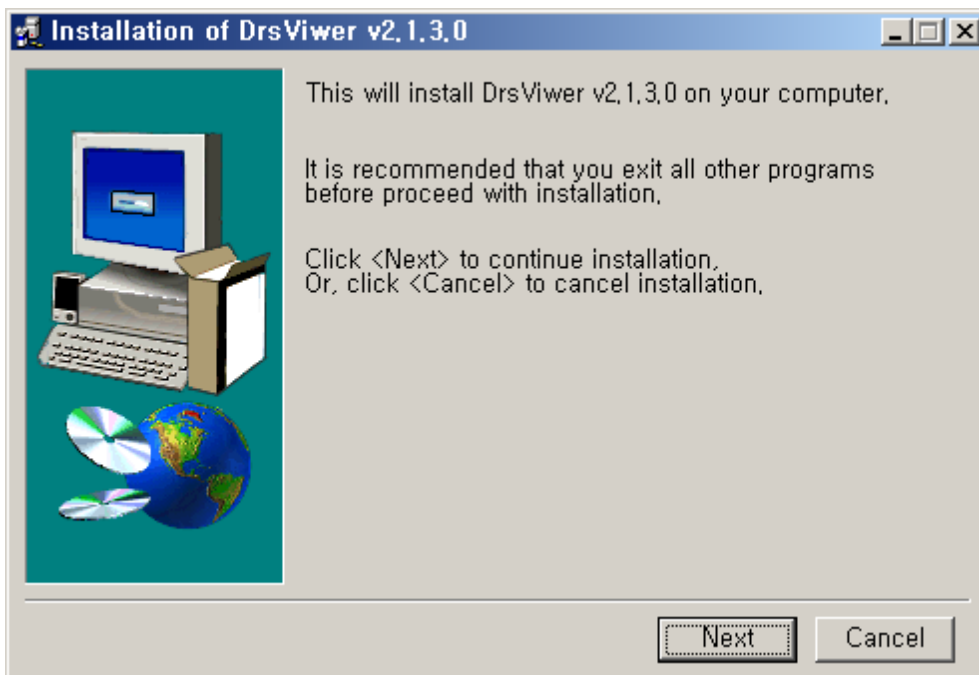
3. Click “Next”


Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

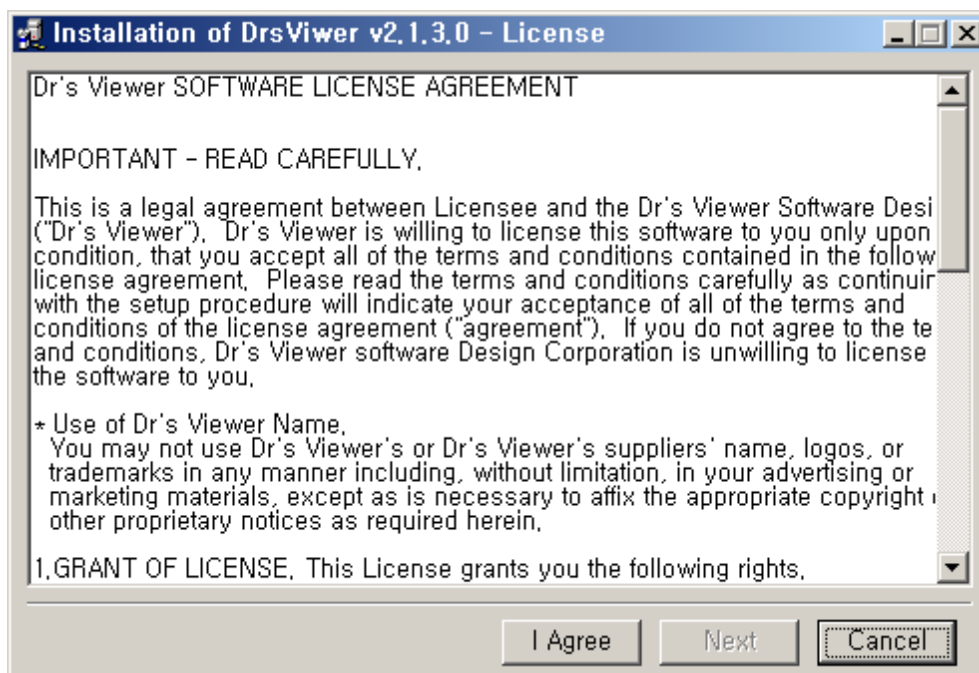4. Click "I Agree" and "Next"

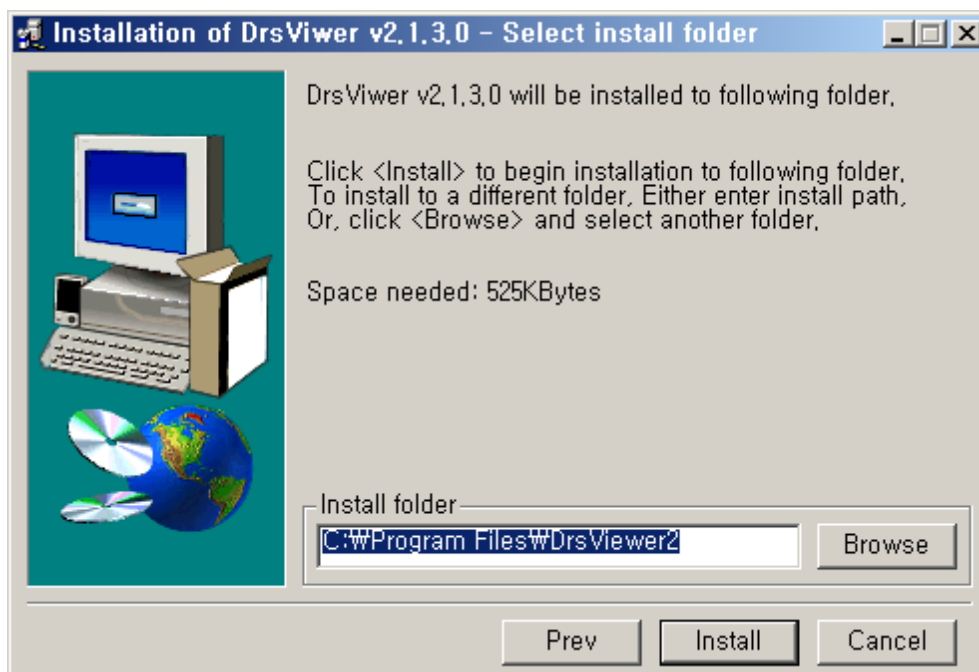
5. Select folder and click "Install"

6. The software is installed well. Click "Ok"

Dr's Viewer 2 Operation Instruction

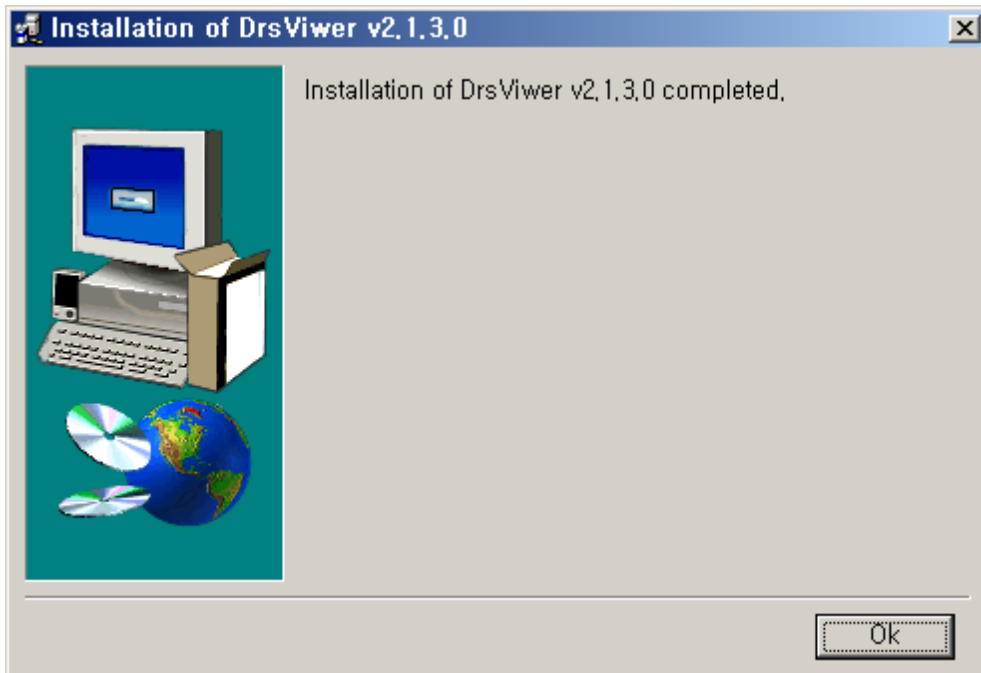Dr's Viewer 2 / Image software

7. After completing steps 1~7, you can find the shortcut on the desktop.


Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

Chapter 3. Overview

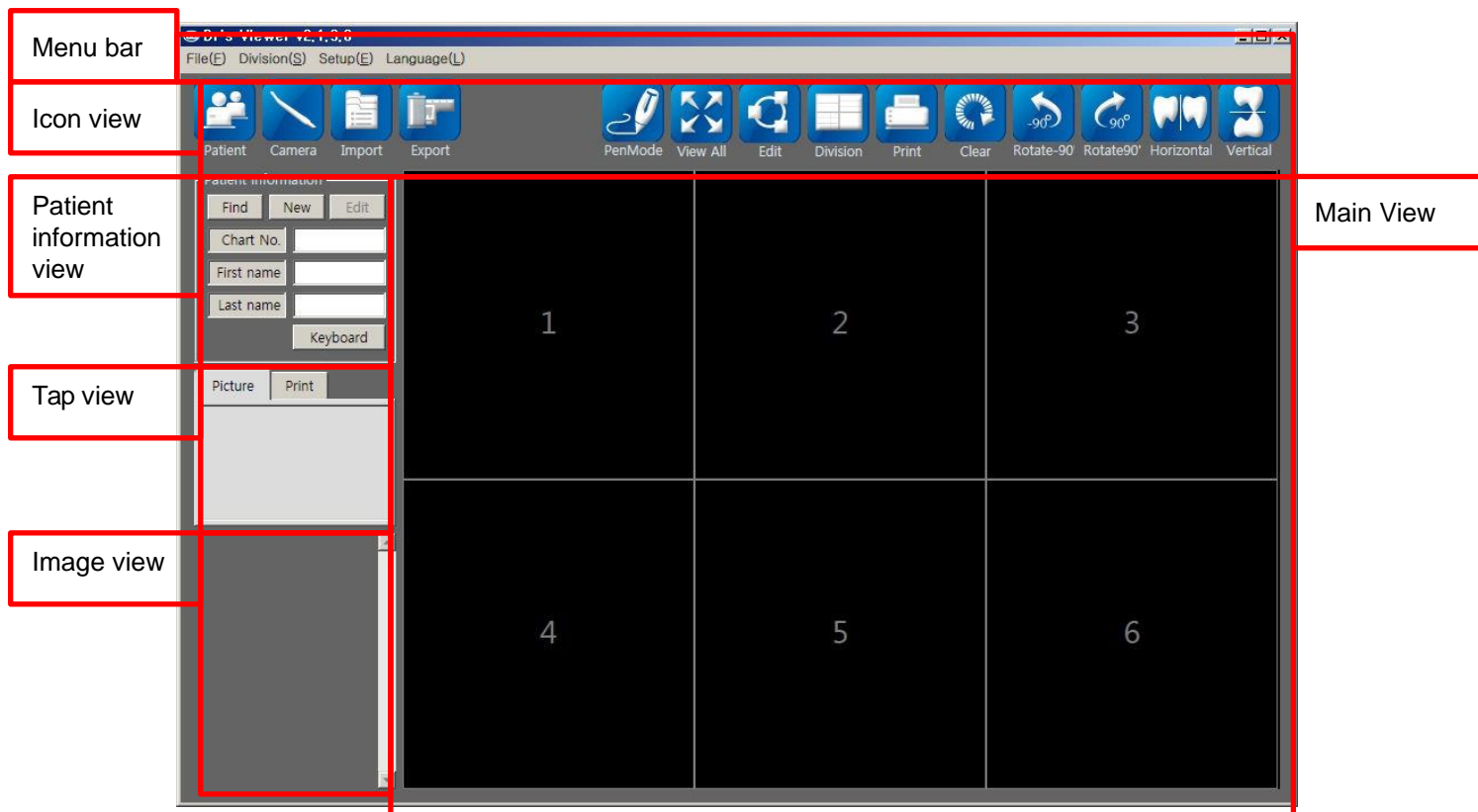After completing the installation, you can run the software. Then you can see the window below.

1. Menu bar

- File(F): Import, Export, Print, Exit
- Division(S): Can adjust image number of main view
- Setup(E): Clinic/DDS setup, Data backup, Import backup
- Language(L): Can select language

2. Icon view

- Patient: Patient registration
- Camera: Go to camera mode
- Import: Image import
- Export: Image export
- Pen Mode<->Normal: Use pen and return to normal.
- View all: See main view image for full screen
- Edit: Image edit
- Division: Dividing main view image

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

- Print: Print the description file with image.
- Clear: Clear all image on the main view
- Rotate -90': Rotate the checked image for -90''
- Rotate 90': Rotate the checked image for 90'
- Horizontal: Reverse the checked image like mirror view.
- Vertical: Reverse the checked image upside and down.

3. Patient information view

- Find: Find registered patient
- New: Patient registration
- Edit: Modify patient data
- Keyboard: Use visual keyboard from Windows

4. Tap view

- Picture: Show date and image number saved.
- Print: Show printed data with date

5. Image view

- Show saved images

6. Main view


- Show selected images.

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software


Chapter 4. Setting

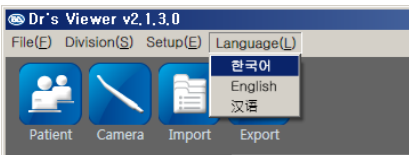1. Select language


- Click Language button and select your language.

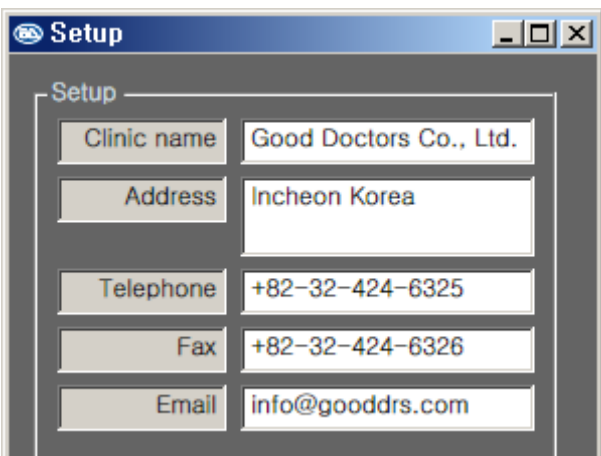2. Clinic registration

- Write your clinic name and other information

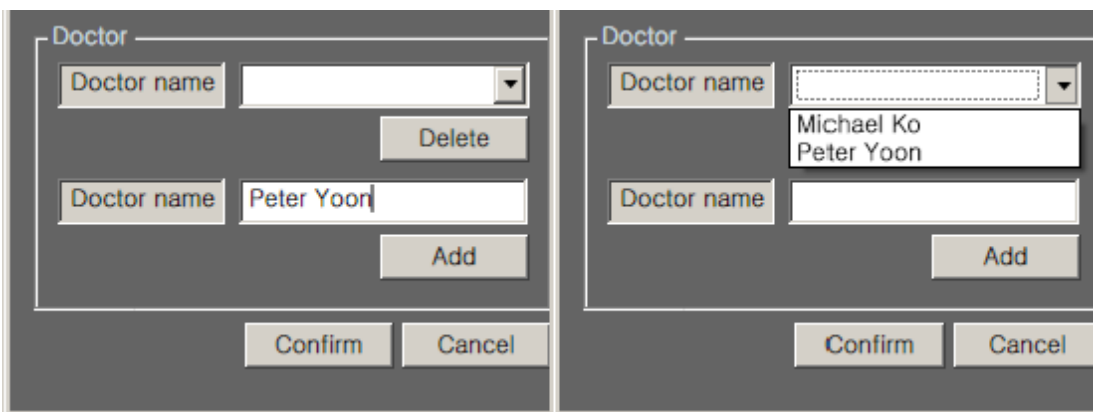3. Doctor registration

- Write the doctor's name and click "Add" button. After that, you will find the doctor's name on the list.

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

Chapter 5. Icon view

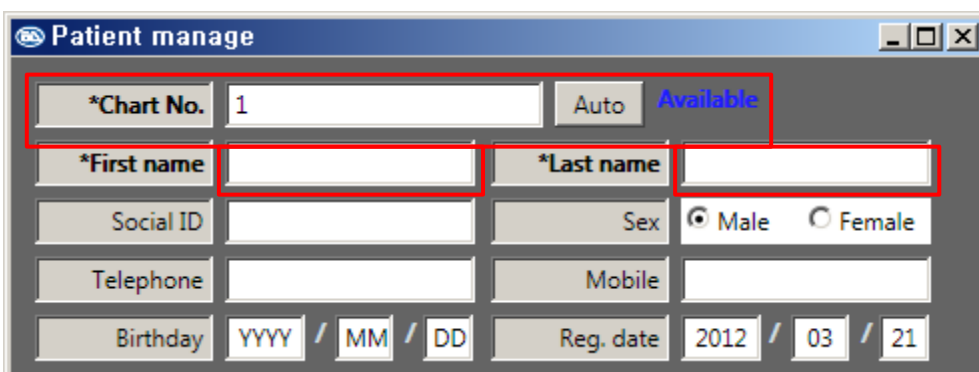
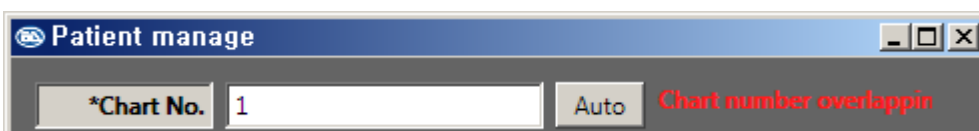
1. Patient

- Insert patient information

1) You can put Chart No. you want. If it is useable, you will see the sign "Available" in blue. If the number is not available, you will see the sign "Chart number overlapping".

* You can click "Auto" button to assign the chart number automatically.

A screenshot of the "Patient manage" window. The window has a blue title bar with the text "Patient manage" and standard window controls. The main area contains several input fields and buttons. The "*Chart No." field contains the number "1". To its right is an "Auto" button and the word "Available" in blue text. Below this are fields for "*First name" and "*Last name", both of which are empty. Further down are fields for "Social ID", "Sex" (with radio buttons for "Male" and "Female"), "Telephone", "Mobile", "Birthday" (with sub-fields for "YYYY", "MM", and "DD"), and "Reg. date" (with sub-fields for "2012", "03", and "21").A second screenshot of the "Patient manage" window, showing the same fields as the first screenshot. However, the "Auto" button is now disabled, and the text "Chart number overlapping" is displayed in red next to it, indicating that the chart number "1" is not available.

2) Write First name, Last name and etc. on the list.

* To save patients information, Chart number, first name and Last name should be written.

2. Camera

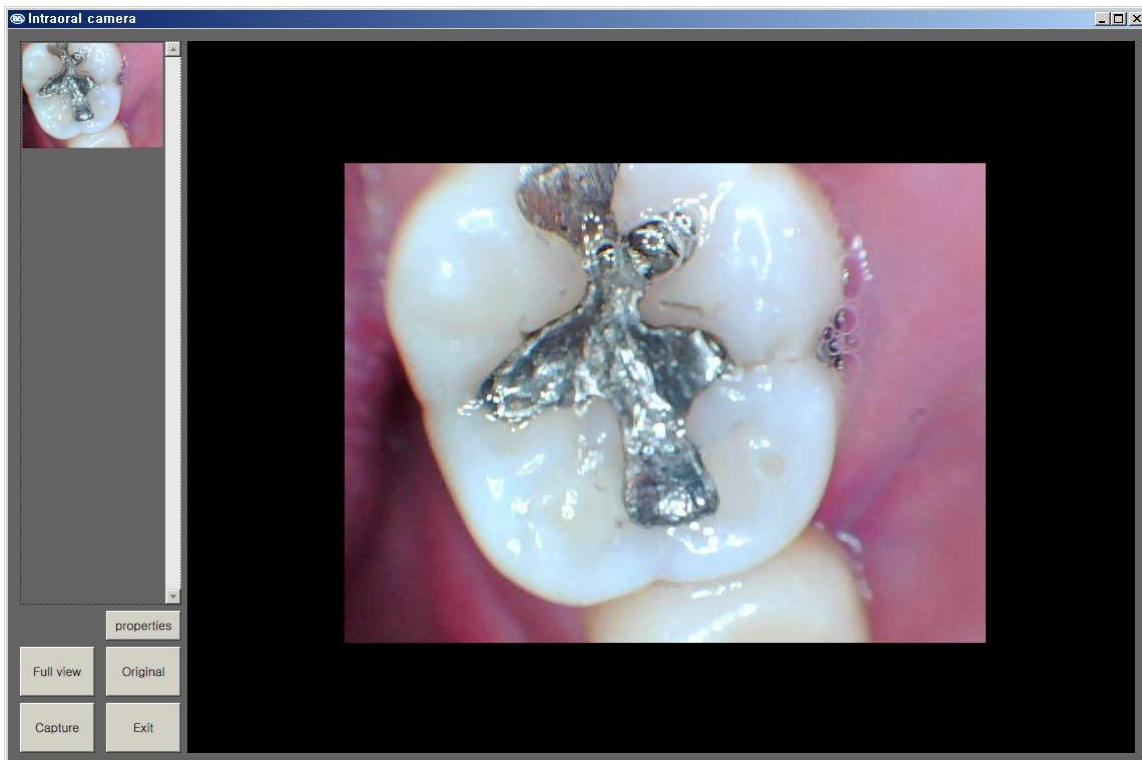
- To run camera function, click Camera icon.

1) When the live image is shown you can capture the image by using the camera, keyboard or mouse


- Camera: Capture button
- Keyboard: Space key
- Mouse: Click "Capture" button

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

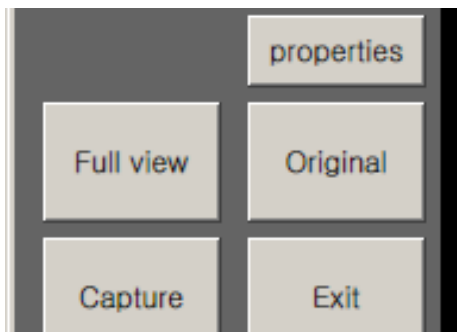2) Button function

Properties: Can adjust brightness, and other contrast of live view.

Full view: Live image is shown by full size.

Original: Live image is shown by original size

Capture: Capture the live image by mouse

Exit: Exit the camera function

3. Import

- User can get image data.

Dr's Viewer 2 Operation Instruction

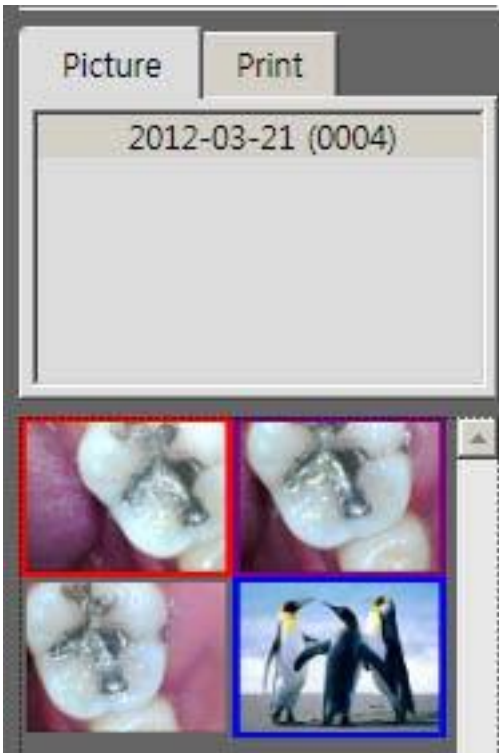
Dr's Viewer 2 / Image software

4. Export

- User can export patient image.

* To export patient images, you should check the image below.

Red colour square check
- Check for export, delete.

Blue colour square check
- It means the image is shown on the main view.

Purple colour square check
- Combined Red and Blue colour square check function.

Dr's Viewer 2 Operation Instruction

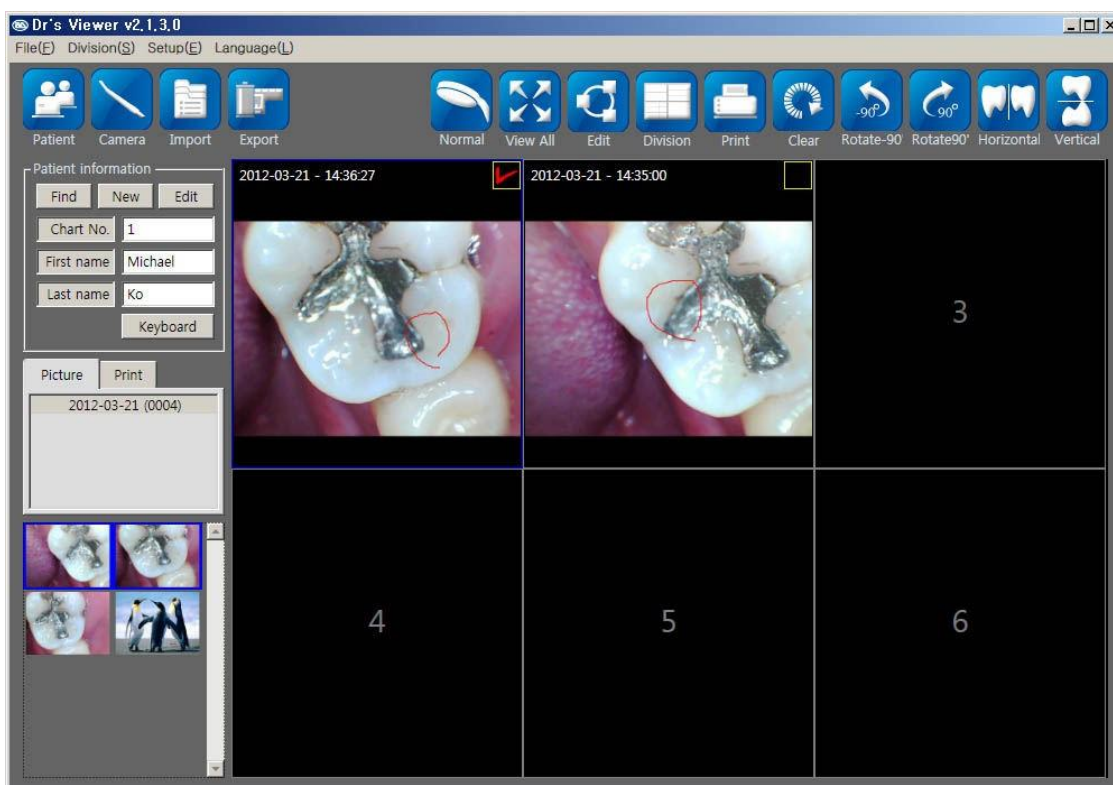
Dr's Viewer 2 / Image software

5. Pen mode

- If you want to write on the main image view, click the pen mode. Then, the icon turns to normal icon, and you can use the pen on the main view.
- Written pen marks on the main view image are not saved.

< Pen mode example >

Dr's Viewer 2 Operation Instruction

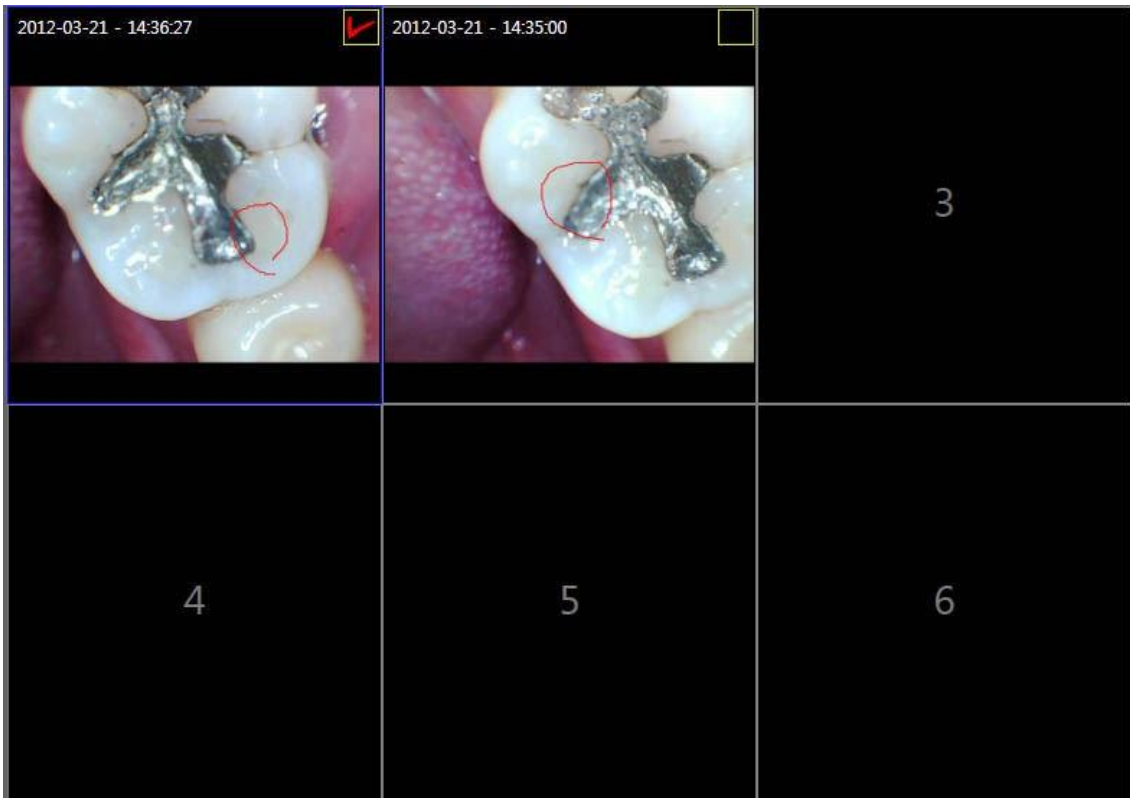
Dr's Viewer 2 / Image software

6 . View All

- Make main view screen to full size.

< View All example >

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

7. Edit

- Can use pen and save the image
- To use Edit mode, the image should be checked

< Edit mode example >

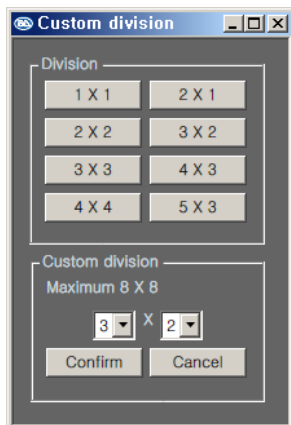
- Can use Pen, Eraser, Zoom-in, Zoom-out, Rotate, and etc. functions.
- Brightness and contrast can be adjusted.
- After modifying it, the image can be saved.

Dr's Viewer 2 Operation Instruction


Dr's Viewer 2 / Image software

8. Division

- Can divide main view from 1 x 1 to 8 x 8.
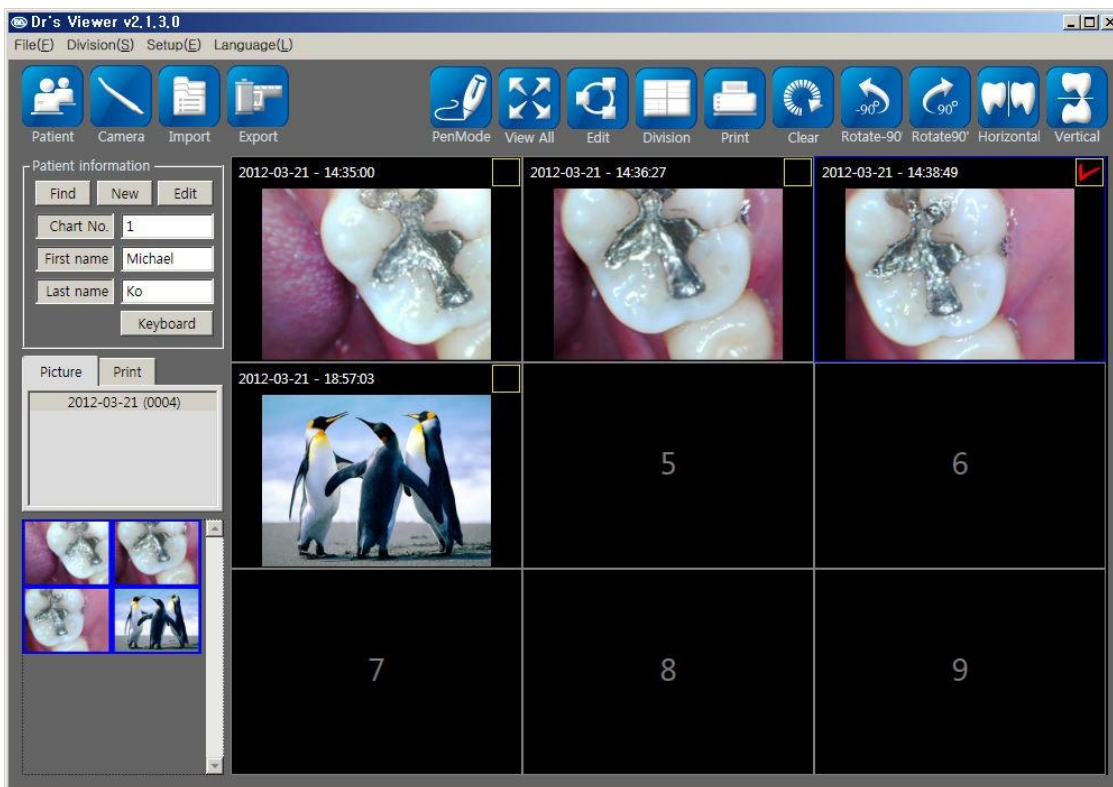
< Example dividing 3 x 3 >

Dr's Viewer 2 Operation Instruction

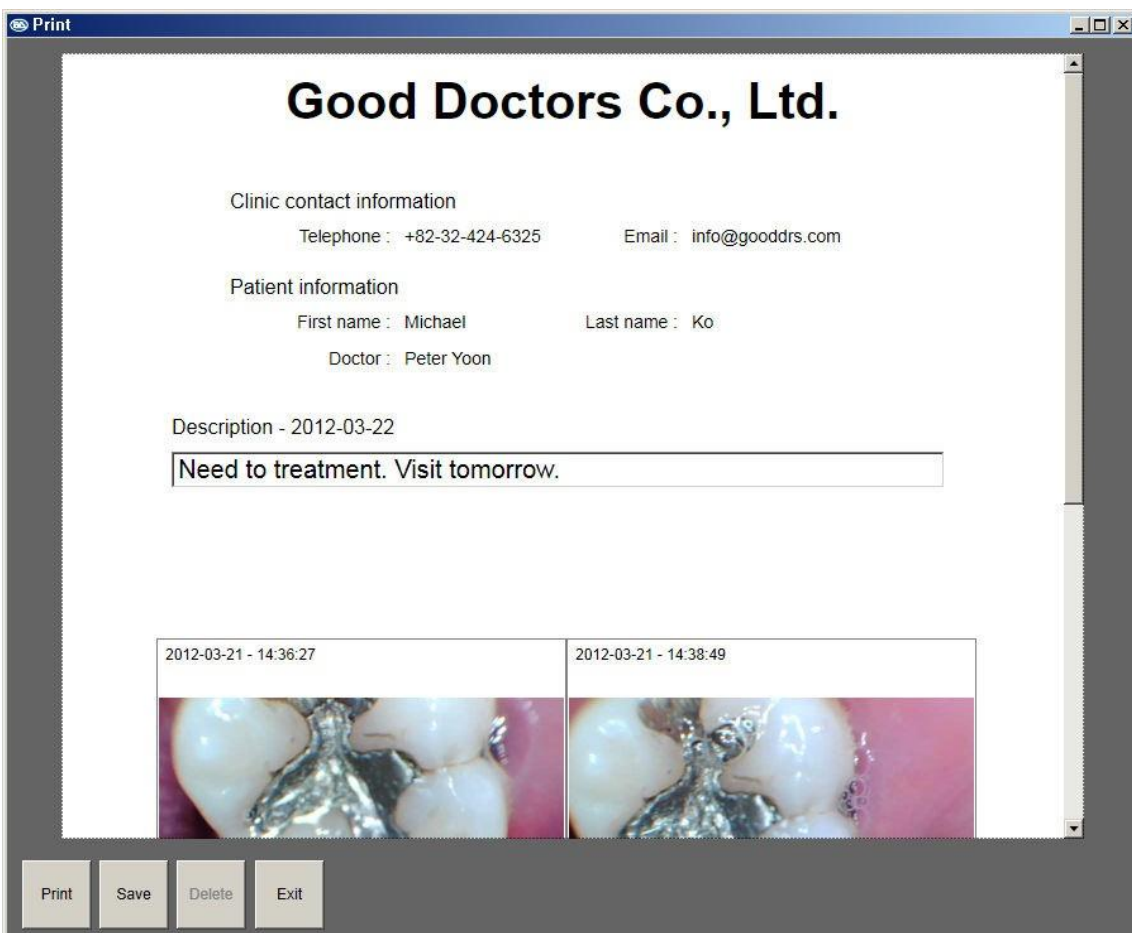
Dr's Viewer 2 / Image software

9. Print

- Can print out images with description
- To print an image, the image should be checked on main view.
- Maximum 4 images can be printed at one time.
- The printing paper can be saved and called back in future.

< Example print mode >

- The saved printed data can be checked at Print tap.

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

10. Other functions


- Clear: Make clear all of image on the main view. The image data is not deleted.
- Rotate -90°: Rotate the checked image for -90°
- Rotate 90°: Rotate the checked image for 90°
- Horizontal: Reverse the checked image like mirror view.
- Vertical: Reverse the checked image upside and down.

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

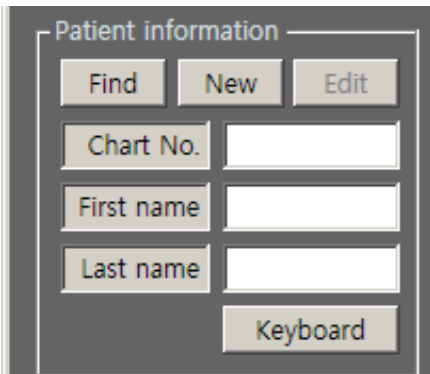Chapter 6. Patient information view


A dialog box titled "Patient information" with three buttons at the top: "Find", "New", and "Edit". Below these are three input fields labeled "Chart No.", "First name", and "Last name". At the bottom right is a "Keyboard" button.

1. Find

- The patient should be checked before using camera mode.

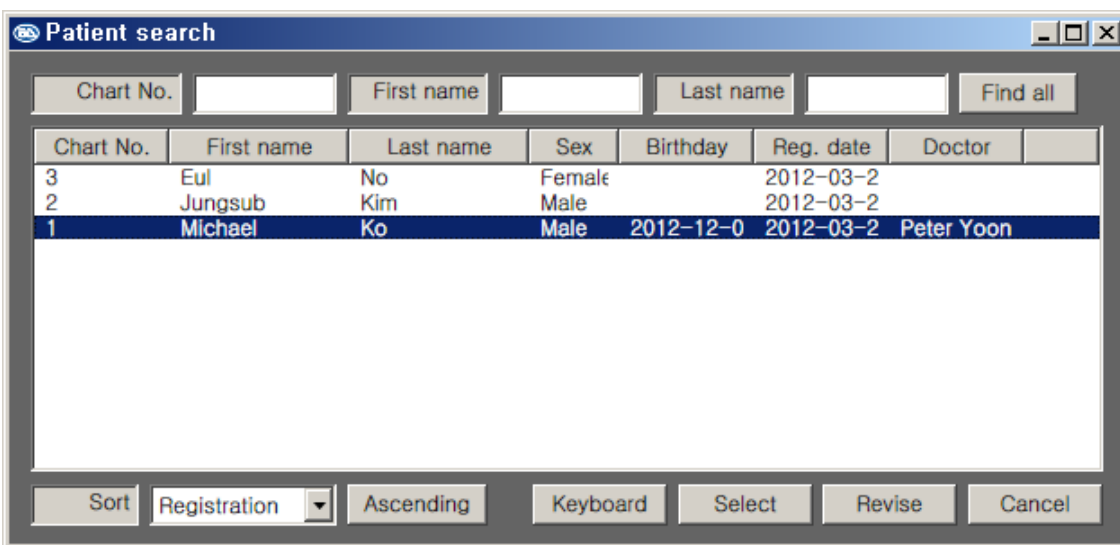1) You can find the saved patient on the list. Select the patient and double click or click "Select" button.

A window titled "Patient search" with search input fields for "Chart No.", "First name", and "Last name", and a "Find all" button. Below is a table with columns: Chart No., First name, Last name, Sex, Birthday, Reg. date, and Doctor. The table contains three rows, with the first row highlighted. At the bottom are buttons for "Sort", "Registration" (dropdown), "Ascending", "Keyboard", "Select", "Revise", and "Cancel".

Chart No.	First name	Last name	Sex	Birthday	Reg. date	Doctor
3	Eul	No	Female		2012-03-2	
2	Jungsub	Kim	Male		2012-03-2	
1	Michael	Ko	Male	2012-12-0	2012-03-2	Peter Yoon

2) If there are many patients, you can sort them by Chart number, First name, Last name or etc.

A control panel with a "Sort" button, a dropdown menu showing "Registration", and an "Ascending" button.

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

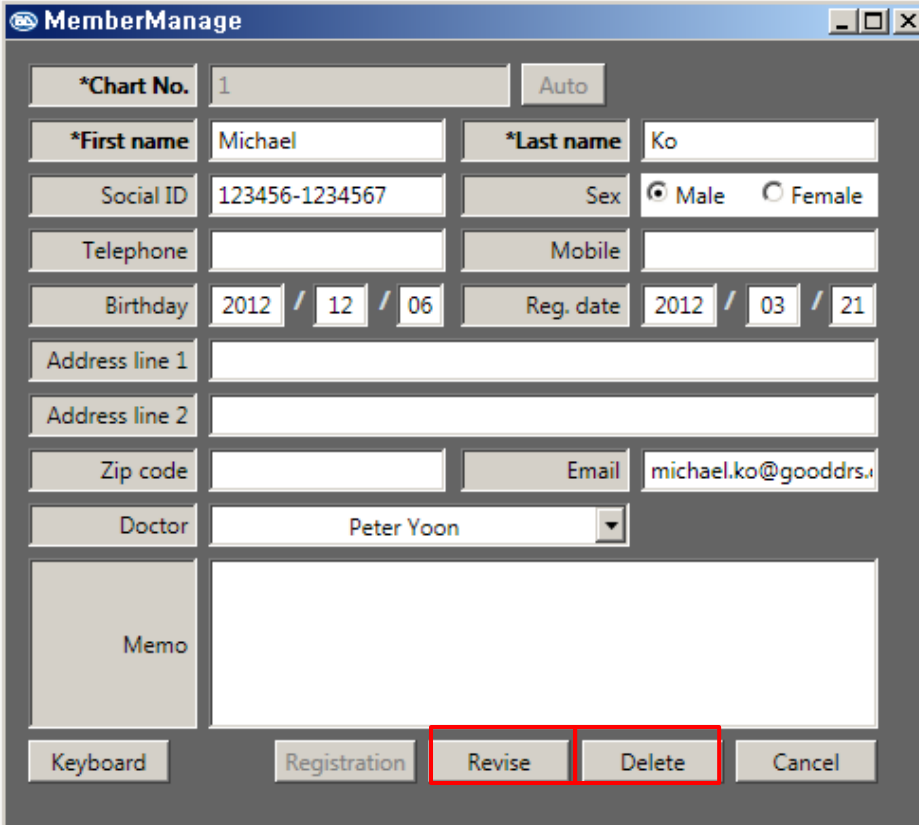
2. New

- Register patient information
- It has the same function as patient icon

3. Edit

- Edit or delete patient information.

1) You can revise all patient information but chart number, or you can delete patient information.

The screenshot shows a software window titled "MemberManage" with a standard Windows title bar. The window contains a form for patient information. The fields are as follows:

*Chart No.	1	Auto	
*First name	Michael	*Last name	Ko
Social ID	123456-1234567	Sex	<input checked="" type="radio"/> Male <input type="radio"/> Female
Telephone		Mobile	
Birthday	2012 / 12 / 06	Reg. date	2012 / 03 / 21
Address line 1			
Address line 2			
Zip code		Email	michael.ko@gooddrs.
Doctor	Peter Yoon		
Memo			

At the bottom of the window, there are five buttons: "Keyboard", "Registration", "Revise", "Delete", and "Cancel". The "Revise" and "Delete" buttons are highlighted with a red border.

4. Keyboard

- It opens visual keyboard from Windows

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software


Chapter 7. Tap view

1. Picture tap

- It shows the number of patient image by date.

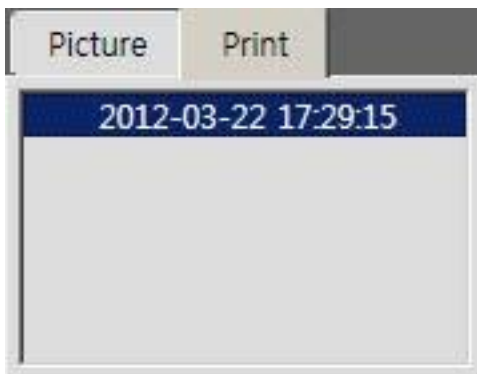2. Print tap

- It shows printed file with its date and time

- Double click shows the printed file.

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

Chapter 8. Image view

- It shows saved image by small size
- Double click or drag an image to main view to see it bigger in the main view
- * No colour square: Just shown in image view
- * Red colour square: the image is selected and can be deleted or exported
- * Blue colour square: shown in image view.
- * Purple colour square: Combined red and blue colour square function.
- * When the image is deleted, the square on the main view is cleared.

Chapter 9. Main view

- It shows bigger images

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

1. Full view

- Double click the image on the main view. Then the image is shown in full screen.
- Number in tap identifies the number of images on the main view
- At full view, can use pen mode and can change its colour and thickness.

2. Other functions

- Click with the right mouse button on the image in main view to visualize more functions:
 - 1) View detail / Revise: Go to Edit mode
 - 2) Full view
 - 3) Clear checked image: Clear checked image of main view. The image data is not deleted.
 - 4) Clear all: Removes all images on the main view. The image is not deleted.
 - 5) Pen Mode: Go to pen mode
 - 6) Line T: Adjust pen thickness.
 - 7) Colour: Adjust pen colour
 - 8) Clear pen checked: Erase pen marks from checked image on the main view
 - 9) Clear pen all: Erase pen marks on all images on the main view
 - 10) Rotate 90': Rotate the checked image for 90'

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software

- 11) Rotate -90': Rotate the checked image for -90'
- 12) Horizontal: Reverse the checked image like mirror view.
- 13) Vertical: Reverse the checked image upside and down.

Dr's Viewer 2 Operation Instruction

Dr's Viewer 2 / Image software


Chapter 10. Data backup

- To save patients data, regular data backup is recommended.


1. Select Setup(E) -> Data backup(B) as follow.

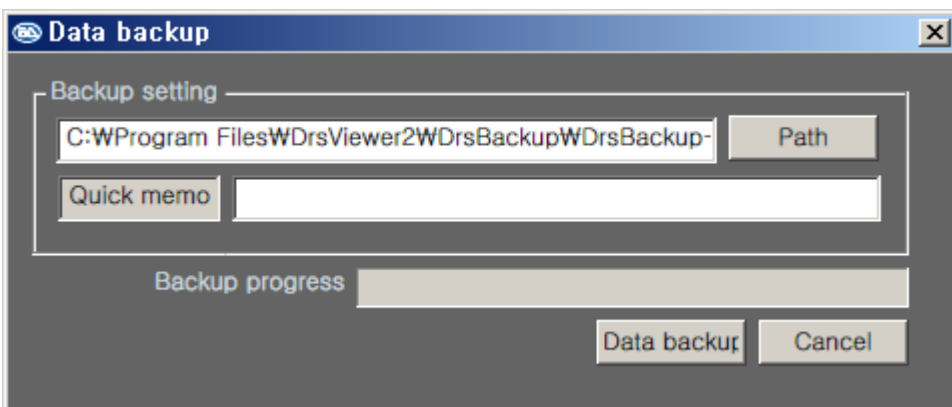2. Choose path to save the backup data and write a memo. Then, click "Data backup" button.

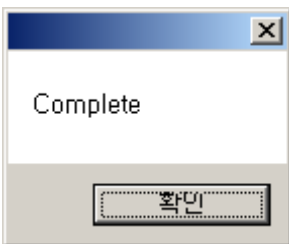3. If you see the below window, the process is done well.

- You can check the backup data as follows:

Dr's Viewer 2 Operation Instruction


Dr's Viewer 2 / Image software

Chapter 11. Import backup


1. Select Setup(E) -> Import backup(L) as follows.

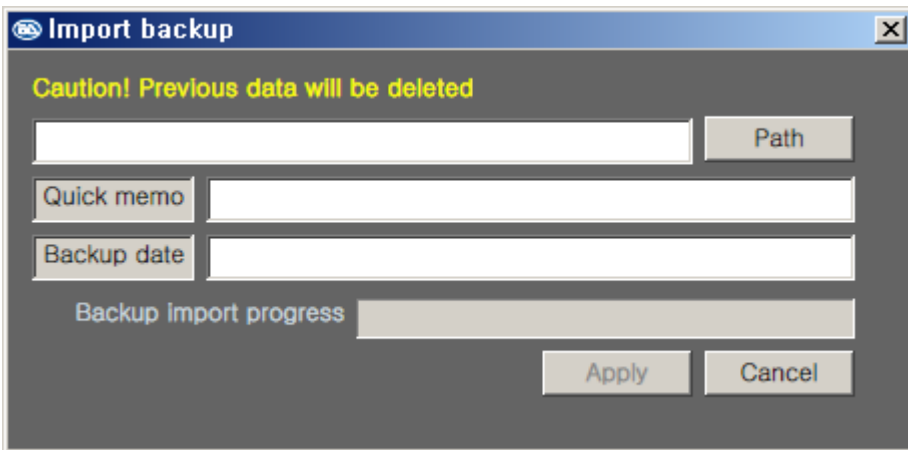
2. Find the backup data folder and select "backup.drb". Then, click "Apply" button

- Check the "backup.drb" file.

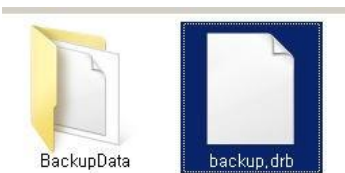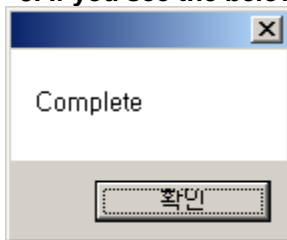3. If you see the below window, the process is done well.

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The image shows two screenshots of the 'Patient manage' window. The top screenshot shows the 'Chart No.' field with the value '1' and the 'Auto' button. The text 'Available' is displayed in blue next to the 'Auto' button. The bottom screenshot shows the same window with the text 'Chart number overlapping' displayed in red next to the 'Auto' button. Red boxes highlight the 'Chart No.' field and the 'Auto' button in both screenshots.

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Miglionico S.r.l.