miglionico
DENTAL EQUIPMENT

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USER MANUAL MIGLIONICO IMAGE

*IMAGES AND VIDEOS ACQUISITION
AND VIEWING SOFTWARE*



TABLE OF CONTENTS

1.	INTRODUCTION.....	23
1.1.	BUTTONS AND FUNCTIONS.....	23
2.	SYMBOLS.....	24
2.1.	UPPER BAR SYMBOLS.....	24
2.2.	LOWER BAR STATUSES.....	24
3.	SETTINGS.....	25
3.1.	SET TIME AND DATE.....	25
3.2.	REPLACE BACKGROUND IMAGE.....	25
3.3.	PEDAL KEYS ON SCREEN.....	26
3.4.	CREATE NEW PATIENT FOLDER.....	27
3.5.	FIND PATIENT FOLDERS.....	27
4.	FUNCTIONS.....	28
4.1.	CAMERA ACTIVATION.....	28
4.2.	AUTO-FOCUS.....	28
4.3.	CAPTURE IMAGE.....	28
4.4.	SAVE PHOTO.....	29
4.5.	RECORD A VIDEO.....	30
4.6.	DELETE ONE OR MORE SAVED FILES.....	31
4.7.	VIEW SAVED IMAGES AND VIDEOS.....	31
4.6.1	VIEW ONE IMAGE FULL SCREEN.....	31
4.6.2	VIEW 4 IMAGES ON SCREEN.....	32
4.6.3	VIEW 6 IMAGES ON SCREEN.....	33
4.6.4	ALTER THE VISUALIZATION ORDER OF THE PHOTOS ON SCREEN.....	33
5.	SHARING SETTINGS.....	34
5.1.	WI-FI CONNECTION.....	34
5.2.	IMAGE SHARING.....	34

1. INTRODUCTION

The image capturing and visualizing software MIGLIONICO IMAGE is designed to interact with the MI-CAM intraoral camera. Thanks to this software, it is possible to capture photos and videos of the oral cavity and save them in separate folders for each patient.

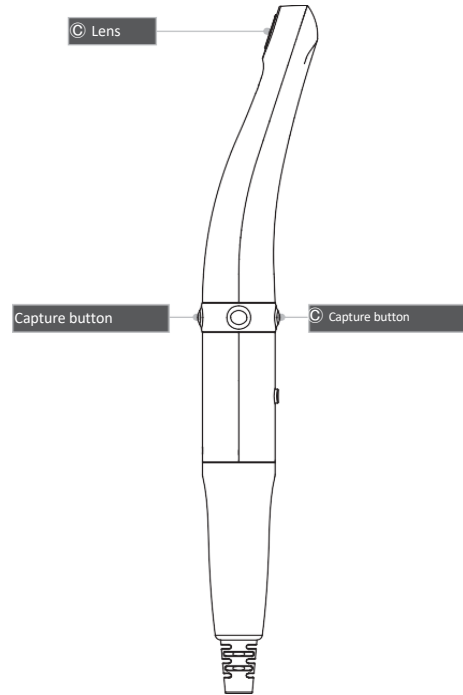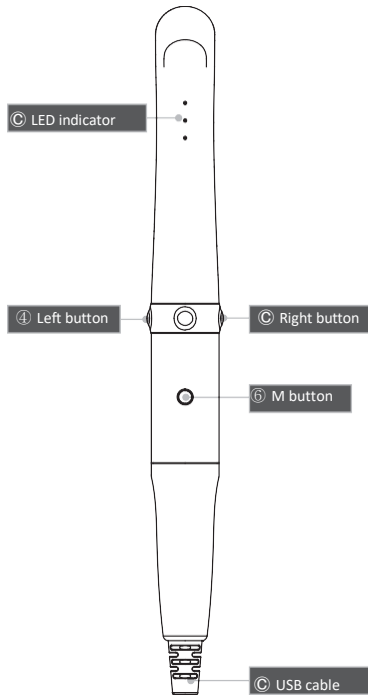1.1. BUTTONS AND FUNCTIONS

1. Lens

- Takes images and send to sensor

2. LED Indicator

- Shows the device status

3. Capture button

- Image capture, focus adjusting

4. Left button

- Mouse left button

5. Right button

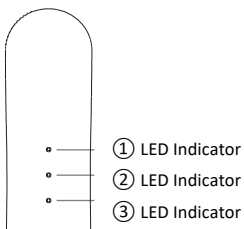
- Mouse right button

6. M button

- Mouse activation

7. USB cable

- Connects the camera to the PC

- ① LED Indicator
- ② LED Indicator
- ③ LED Indicator

LED Indicator	Status	LED colour			
		●	●	●	OFF
① LED	Black and white mode	Normal mode	-	-	Black and white mode
② LED	Colour impression	Original	Standard	Warm	-
③ LED	Auto-focus	Semi auto-focus	Full Auto-focus	-	-

For more information about the successful operation, daily maintenance and safety for the use of this device, please read the MI-CAM user manual supplied with the camera.

2. SYMBOLS

2.1. UPPER BAR SYMBOLS

	<i>SOFTWARE SETTINGS</i> To set the basic software settings, see par. 3.
	<i>WI-FI CONNECTION</i> To configure your network, see par. 5.1.
	<i>EJECT SUB DRIVE</i> (if present)
	<i>ADD NEW PATIENT FOLDER</i> To create a new patient folder, see par. 3.4.
	<i>FIND PATIENT FOLDER</i> To find an existent folder, see par. 3.5.
	<i>MAIN ROOT</i> If selected, it allows you to save images in the main root, that is outside all the patients folders.
	<i>VIDEO MODE</i> If selected, the camera will go into video mode (see par. 4.4)
	<i>RESET DEFAULT VIEW</i> (when visualizing more images on screen) Resets the order of the images on the screen to the default view
	<i>VIEW ONE IMAGE ON SCREEN</i> See par. 4.6.1.
	<i>VIEW 4 IMAGES ON SCREEN</i> See par. 4.6.2.
	<i>VIEW 6 IMAGES ON SCREEN</i> See par. 4.6.3.
	<i>HOME</i> Go back to the main screen
	<i>TURN OFF PC</i>


2.2. LOWER BAR STATUSES

 USB or  SD	<i>USB/SD DEVICE CONNECTED</i> (green) or <i>ABSENT</i> (red)
 Serial	<i>CHECK SERIAL</i>
 Pedal Sx	<i>FOOTPEDAL LEVER TO THE LEFT</i> (green if ON, red if OFF)
 Pedal Down	<i>FOOTPEDAL LEVER DOWN</i> (green if ON, red if OFF)
 Active Camera	<i>CAMERA ON/OFF</i>
 WiFi: 192.168.168	<i>IP</i> (green if connected to wi-fi, red if disconnected)
PATH:\192.168.222\SD	<i>IMAGE SHARING PATH</i>

3. SETTINGS

3.1. SET TIME AND DATE

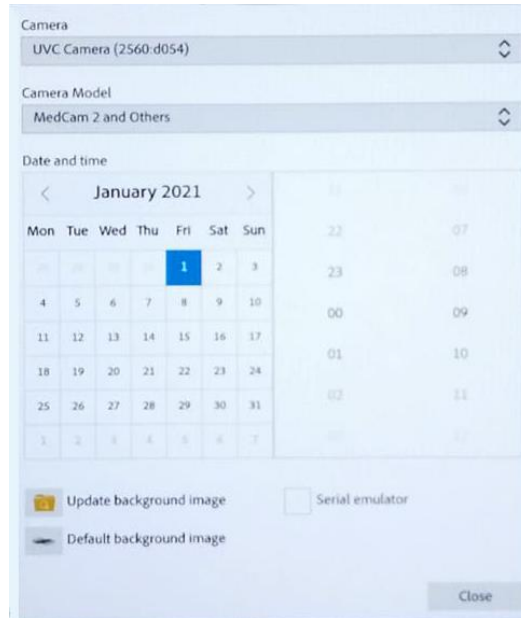
To set time and date, click on the setting icon  on the top left corner and a window showing a section to set time and date will appear.

Set time and date and click on CLOSE at the bottom right corner to save these settings. If you accidentally click on the settings button, click on CLOSE to cancel the action.

3.2. REPLACE BACKGROUND IMAGE


To replace the background image of the home screen, click on the setting icon  on the top left corner and a window showing a section to set time and date will appear.

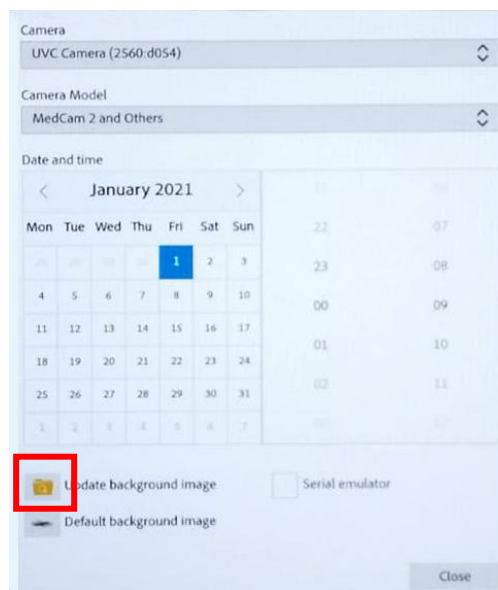USER MANUAL MIGLIONICO IMAGE


Then, click on the "UPDATE BACKGROUND IMAGE" icon circled in red and a folder showing the images saved in the main root of the SD device connected to the unit will open. To select an image, click on the chosen image, then click on CLOSE in the time and date setting window and the selected image will automatically appear as the background of the home screen.

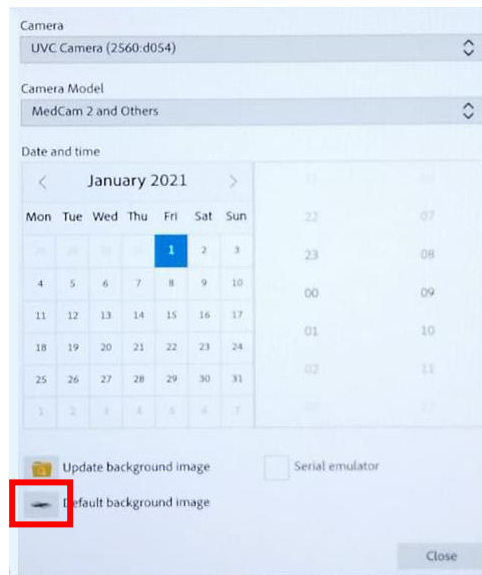To reset the default background image, go back to the settings menu and click on DEFAULT BACKGROUND IMAGE and then CLOSE.


3.3. PEDAL KEYS ON SCREEN

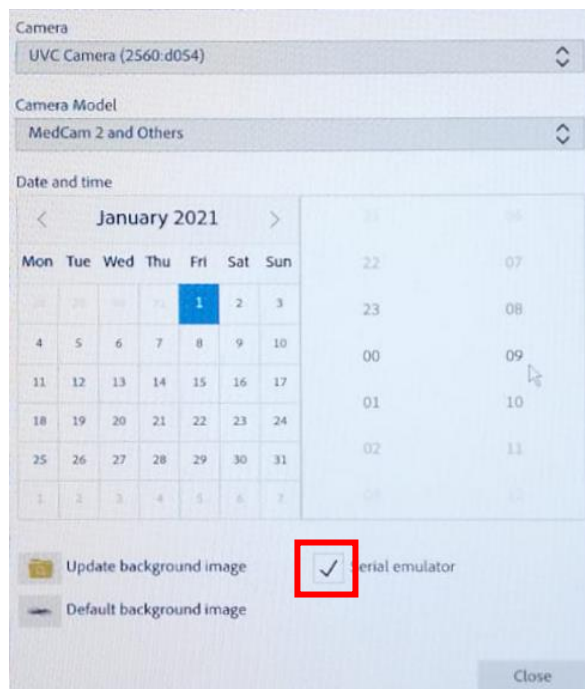To view the foot-pedal functions on screen and use them through the mouse, click on the setting icon and a window showing a section to set time and date will appear.

Tick the box SERIAL EMULATOR by clicking it once and then click CLOSE.

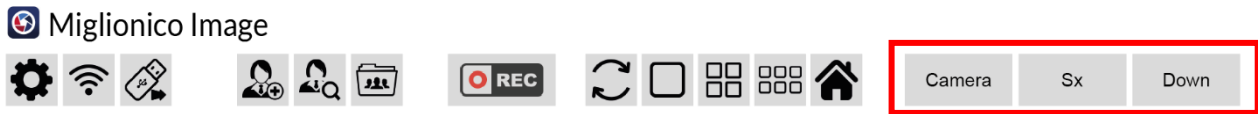You will see in the upper bar 3 buttons which emulate the foot-pedal functions and can be used though the mouse:

CAMERA: captures the image
lever down

SX: foot-pedal lever on the right

DOWN: foot-pedal

3.4. CREATE NEW PATIENT FOLDER

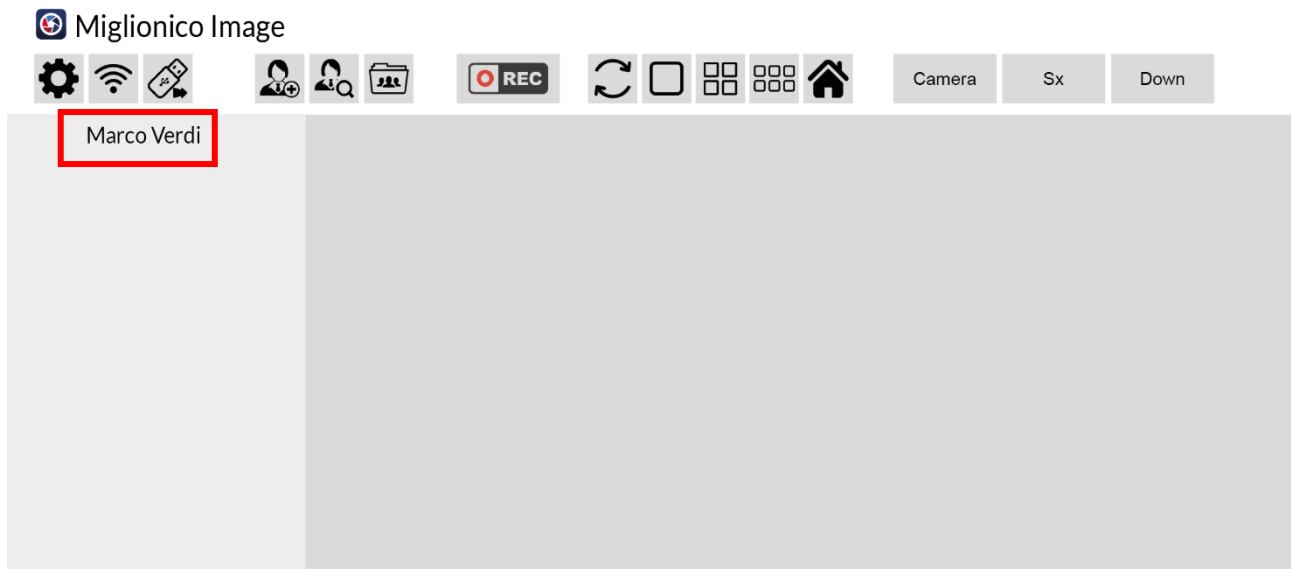
To create a new patient folder, click on the icon  on the upper bar.

A dialog box with a title 'Name' and a single-line text input field. A 'Cancel' button is located at the bottom right of the dialog.

Then, a blank tab to insert the name of the patient will appear.

Insert the patient's name and click on ADD to create the folder.

The folder will automatically open and the name of the patient will be shown at the top of the preview gallery on the left.


3.5. FIND PATIENT FOLDERS

To find and open an already existing patient folder, click on the icon  on the upper bar.

Then, a window showing a list of the existing folders will open. Double click on the chosen folder to open it.

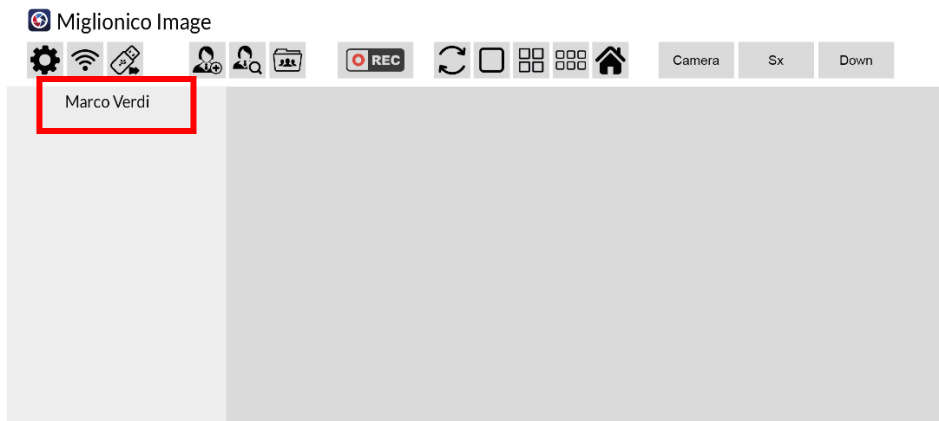The folder will open and the name of the patient will be shown at the top of the preview gallery on the left.


4. FUNCTIONS

4.1. CAMERA ACTIVATION

After selecting the camera from its support, wait 1 sec and the live image will appear automatically on screen.

4.2. AUTO-FOCUS

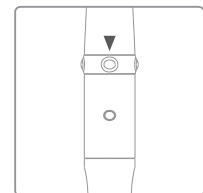
When the camera is in semi auto-focus mode, a light pressure on the button indicated by the arrow will be sufficient to focus the image, while in full auto-focus mode, the camera will automatically focus and capture the image with a light pressure on the same button.


4.3. CAPTURE IMAGE

To capture the image, it is possible to use the foot-pedal or the buttons on the camera.

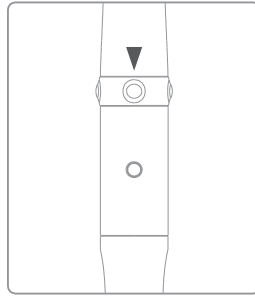
- **CAMERA:** press the capture button on the camera.

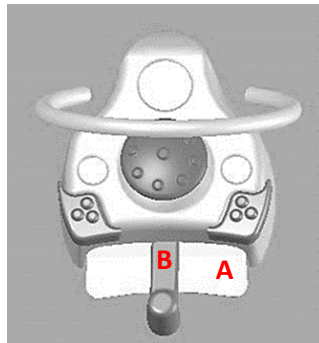- **FOOT-PEDAL:** push (A) the steel lever at the base of the pedal downwards.

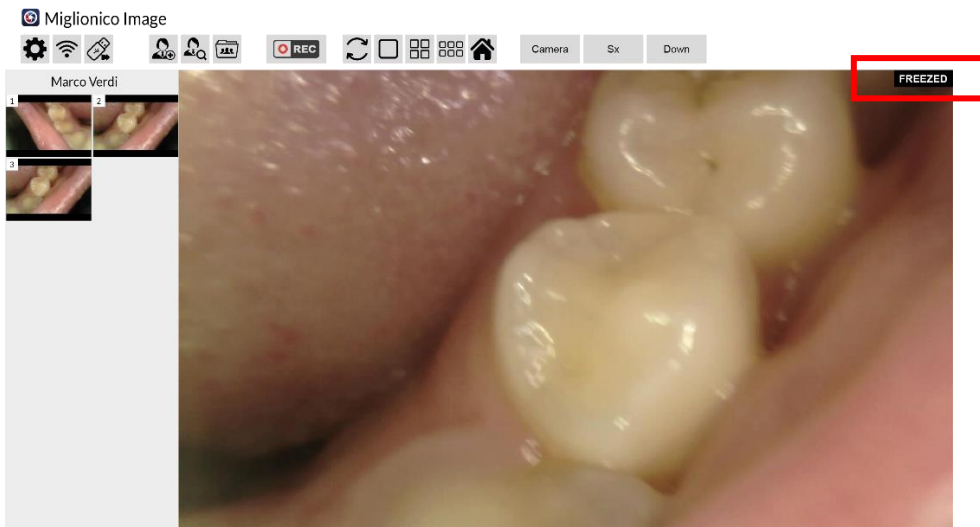Then, the captured image will be shown on screen with a label "FREEZED" at the top right corner.


4.4. SAVE PHOTO

When the image is freezed and you want to save it, move the foot-pedal lever (B) to the left. In this way, the photo is saved and the camera returns to live mode. The preview of the memorized photo will appear in the preview gallery on the left.

If you want to discard the photo, push again the foot-pedal lever (A) downwards. In this way, the photo is discarded and the camera returns to live mode.

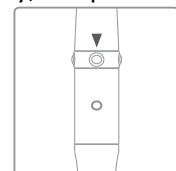
To go back to live mode and discard the photo using the camera buttons, press and hold the capture button on the camera.

4.5. RECORD A VIDEO


To record a video, follow the following steps:

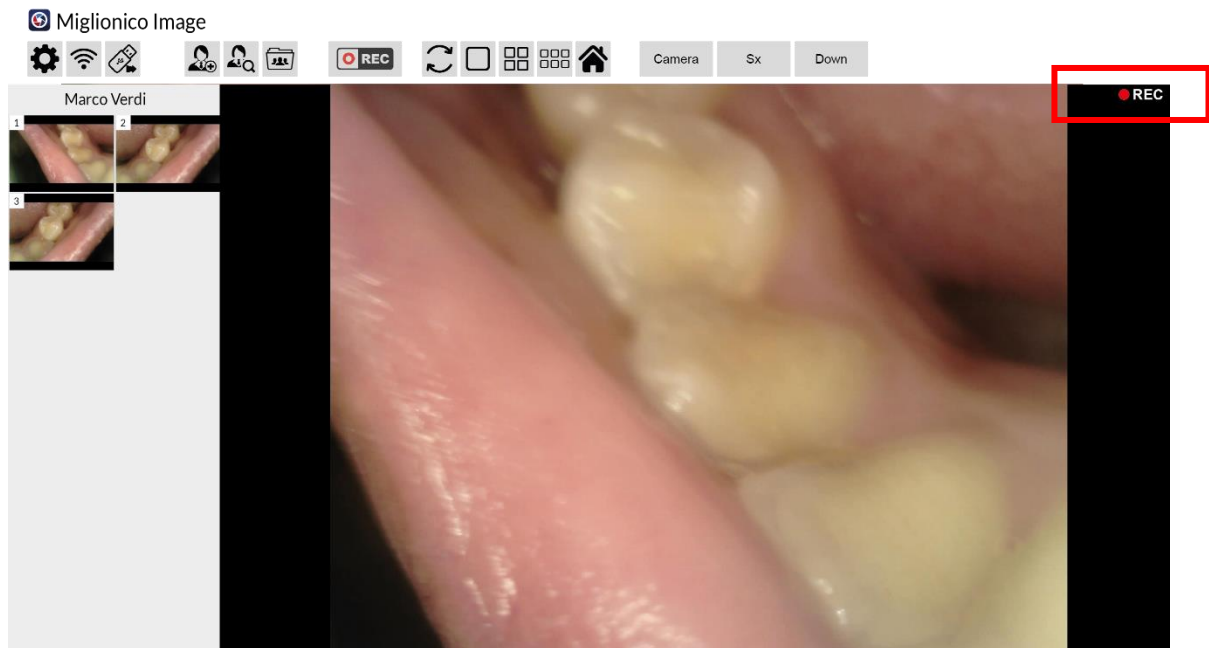- Click the icon  to go to video mode;
- Remove the camera from its holder;
- Push the foot-pedal lever (A) downwards to start recording;
- Wait until the REC symbol is shown at the top right corner of the recording panel;

- Push the foot-pedal lever (A) downwards again to stop recording.

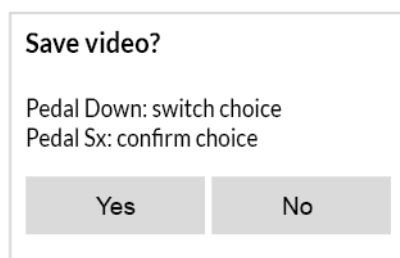After you have done that, a pop-up to save or discard the recorded video will appear.


Push the lever (A) at the base of the pedal to choose between YES and NO; move the pedal lever (B) to the left to confirm your choice.

To return to photo mode, put the camera back in its holder and click again on the icon .

4.6. DELETE ONE OR MORE SAVED FILES

To delete an image or a video, you can use the foot-pedal or the mouse.

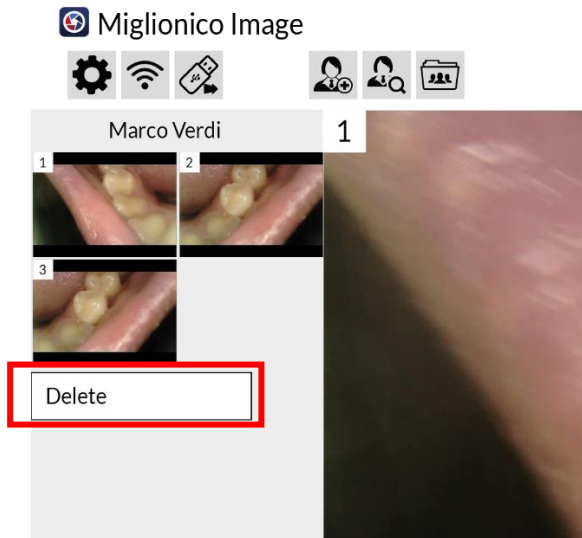
- **MOUSE:** You can delete one image/video at a time using the mouse.
Click on the preview of the image/video in the preview gallery with the right mouse button.

A pop-up marked DELETE will appear.

By clicking on DELETE, the image/video will be deleted.

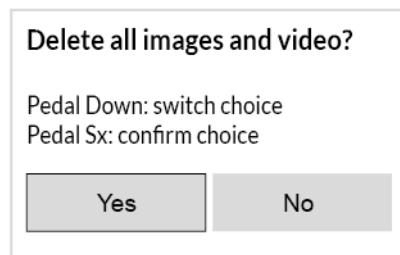- **PEDAL:** you can delete all the images/videos of a folder at a time. Push and hold the lever (A) at the base of the pedal. Then, a pop-up to confirm or cancel the action of eliminating all the images and videos will appear.

Push the lever (A) at the base of the pedal to choose between YES and NO; move the pedal lever (B) to the left to confirm your choice.

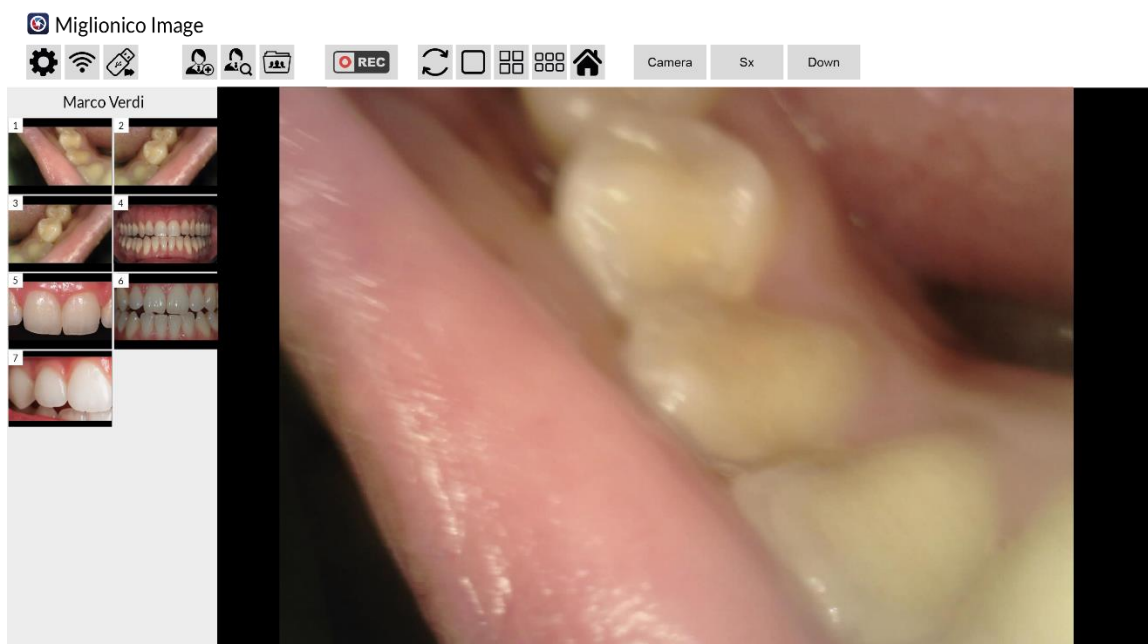
4.7. VIEW SAVED IMAGES AND VIDEOS

To view the saved images and videos, you can use the foot-pedal or the mouse.

4.6.1 VIEW ONE IMAGE FULL SCREEN

PEDAL: Take the camera from its holder and the camera will go in live mode. Then, move and hold the foot-pedal lever (B) to the left until the first image is shown on the screen (approx. 3 secs). If you move again the foot-pedal lever (B) to the left, you will view the next image and so on. To return to the live mode, push the foot-pedal lever (A) downwards.

MOUSE: Click on the icon  on the upper bar to view the image full screen. To pass to another photo, double-click on the chosen photo in the preview gallery.

4.6.2 VIEW 4 IMAGES ON SCREEN

PEDAL: Take the camera from its holder and the camera will go in live mode. Then, move and hold the foot-pedal lever (B) to the left. After the 1st picture full screen is shown, keep holding the lever until 4 images are shown and then release the lever. If you move again the foot-pedal lever (B) to the left, you will view the next image and so on. To return to the live mode, push the foot-pedal lever (A) downwards.

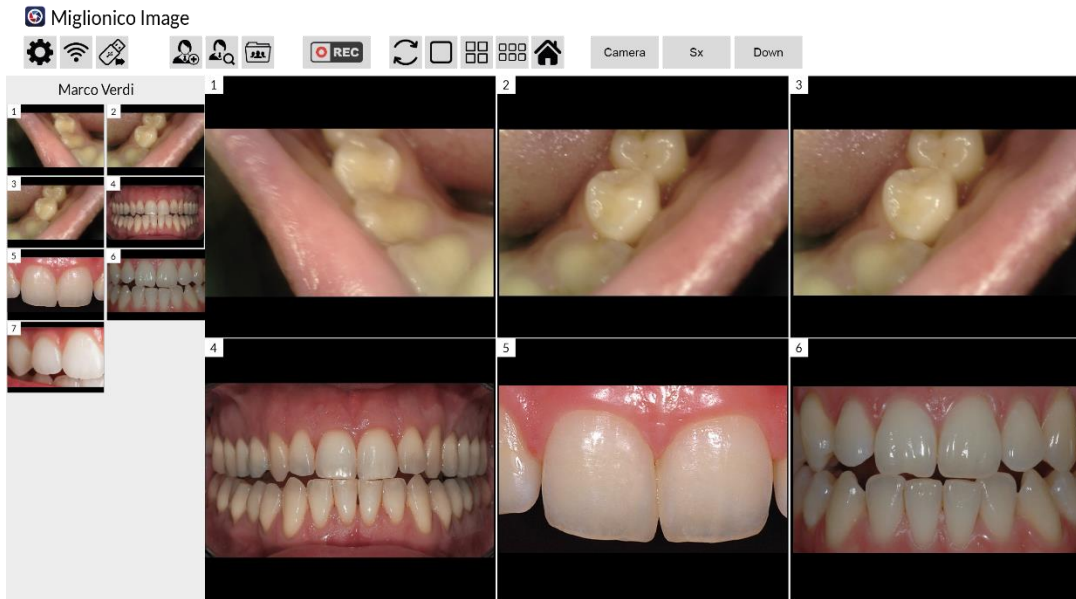
MOUSE: Click on the icon  on the upper bar to view 4 images at the same time on screen.

4.6.3 VIEW 6 IMAGES ON SCREEN


PEDAL: Take the camera from its holder and the camera will go in live mode. Then, move and hold the foot-pedal lever (B) to the left. After the 1st picture full screen is shown, keep holding the lever until 4 images and then 6 are shown. Then, release the lever. If you move again the foot-pedal lever (B) to the left, you will view the next image and so on. To return to the live mode, push the foot-pedal lever (A) downwards.

MOUSE: Click on the icon  on the upper bar to view 6 images at the same time on screen.

4.6.4 ALTER THE VISUALIZATION ORDER OF THE PHOTOS ON SCREEN

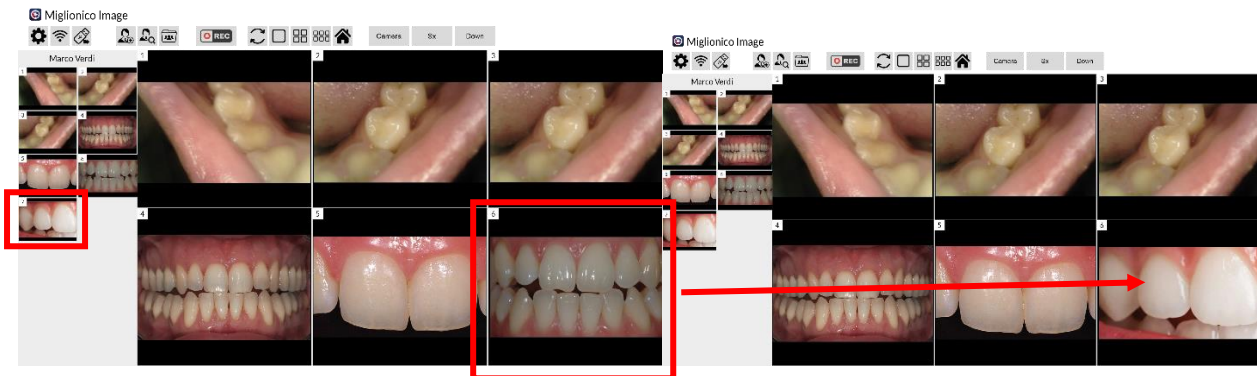To alter the visualisation order of the photos on screen, use the mouse to click on the photo you want to insert from the preview gallery and then click on the photo to replace on the main screen.

To reset the default visualization order, click on 

5. SHARING SETTINGS

5.1. WI-FI CONNECTION

Click on the icon  on the upper bar and a pop-up will appear. Choose the wi-fi network and insert the password. Click on SAVE to connect.

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Password
8 characters at least

Save Close

Once connected, the IP will be shown in the lower bar:

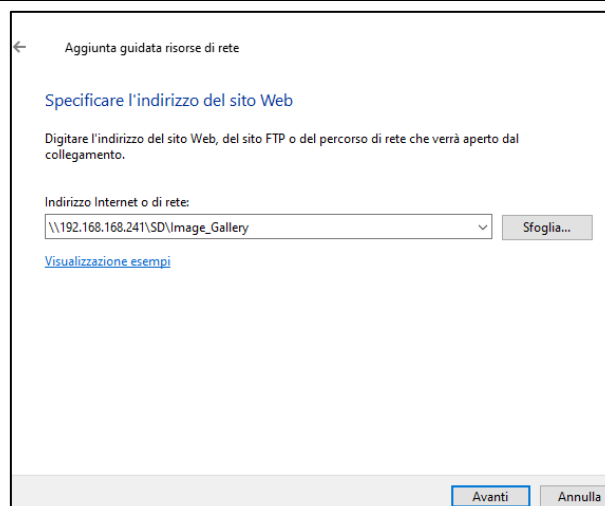
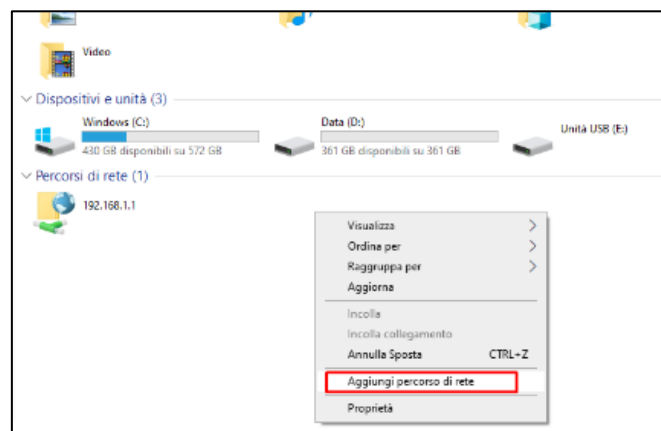
 WiFi: 192.168.168.222

5.2. IMAGE SHARING



1. Connect to the wi-fi network of the clinic by following the procedure explained in par. 5.1.
2. Select the PC through which you want to access the contents of the capture card.

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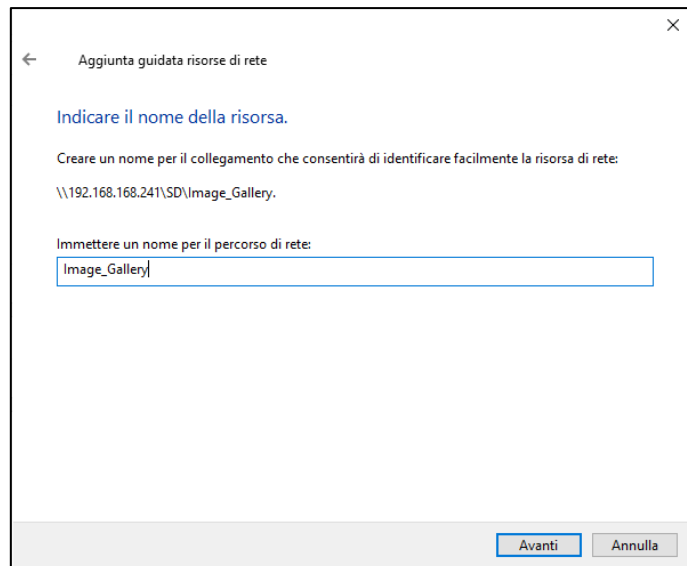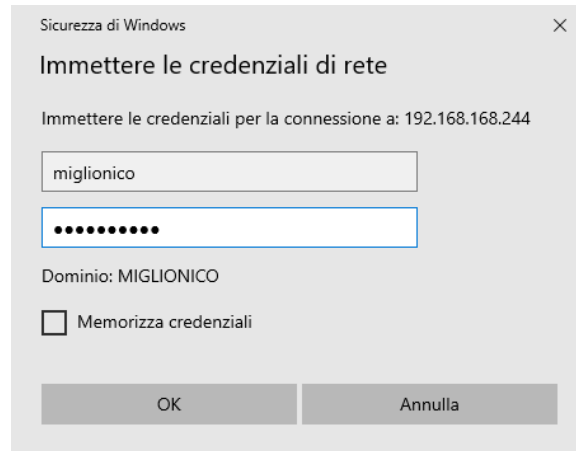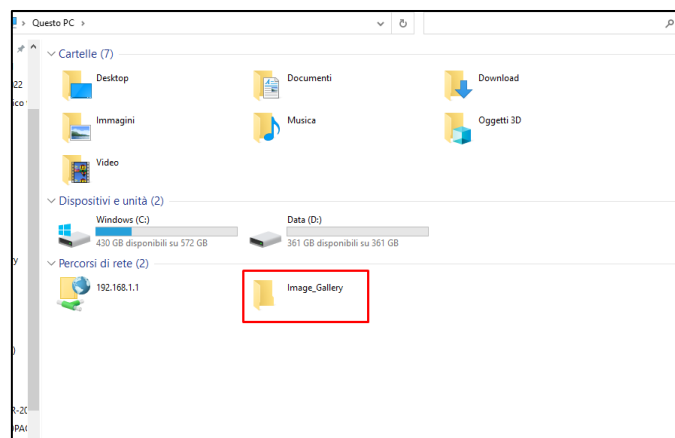
3. Click on next and finish. The network resource will be created on this PC.

4. The network path on which the images will be saved will be shown in the lower bar
PATH: \\192.168.168.222\SD\Image_Gallery (0%)

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
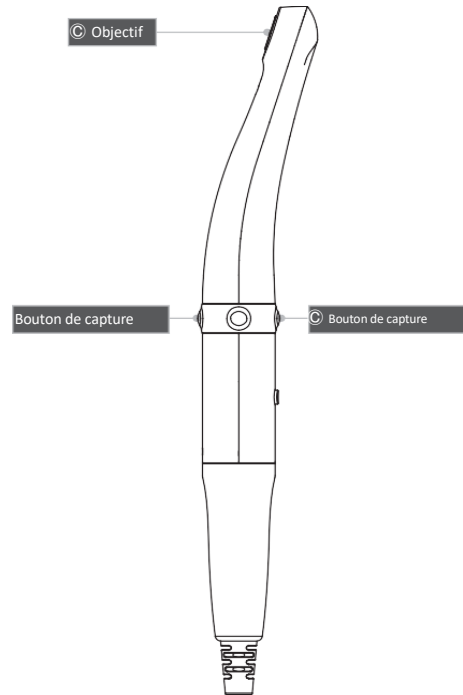
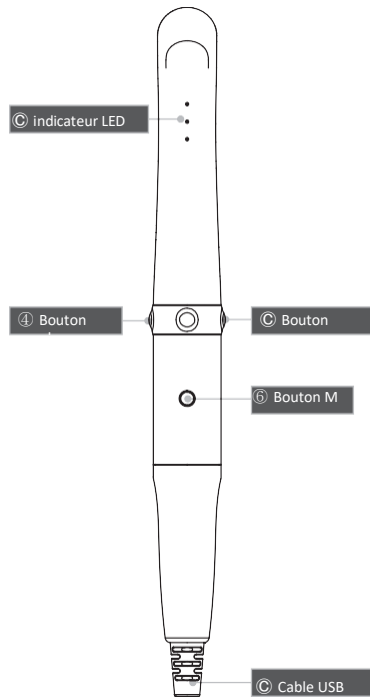
1. Objectif

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2. Indicateur LED

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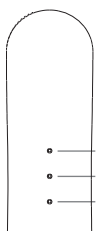- ① Indicateur LED
- ② Indicateur LED
- ③ Indicateur LED

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Miglionico S.r.l.